CORIUS SEAL™

Contents: 1 PCS

Catalogue
Number

Made in USA

Non sterile

Do not use
if package
is damaged

Do not
re-use

Does not
contains or
presence of
natural
rubber
latex

Use by
date

104°F
40°C
41°F
5°C

Medical
Device

Batch
Number

Unique
Device
Identifier

Consult
instructions
for use

Importer

Authorized
Representative
in the EC

Safeguard Technologies Ltd.
Willow Grove Delgany,
Co Wicklow, A63 XY89, Ireland

Safeguard Medical Technologies Ltd.
The Old Rectory, Hope Under Dinmore
Herefordshire, HR6 0PW, United Kingdom

Intended Use

The Corius Seal™ wound dressing is intended to cover & protect acute wounds and absorb exudate.

User Profile

The Corius Seal™ is indicated for use by laypersons and medical professionals.

Patient Profile

The Corius Seal™ is indicated for use on patients with acute wounds.

Clinical Benefits

To cover and protect an acute wound, which helps to prevent further harm and lowers risk of infection, whilst absorbing exudate.

Performance Characteristics

- Hydrogel adheres to skin but is easily removed.
- Hydrogel absorbs wound exudate.

Product Claims

- Removable plastic liners and non-woven spunlace backing aids with easy application and manipulation.
- Dressing is made from breathable materials.
- Natural Rubber Latex Free.
- Robust performance under extreme environmental conditions.

Contraindications

- Do not use on chronic wounds.
- Consult a clinical physician before applying to infected or heavily bleeding wounds, or if skin reaction occurs.


Residual Risks/Warnings

- Excessively greasy skin will reduce the ability of the adhesive to function. If the patient's skin is suspected to be saturated with lipid-based lotion, clean with a skin cleansing wet wipe prior to application.
- If difficulty is experienced on removing the dressing, it should be irrigated or soaked with water or sterile saline solution.
- In case of intolerance, allergic reaction or irritation reaction to the dressing, remove and clean the area carefully, and seek medical advice.
- Dispose of the dressing appropriately or as per any local guidelines. Improper disposal risks cross-contamination.
- This product is intended for single use only. Reuse will risk cross-contamination and loss of product functionality.
- Do not use after expiry date. Using past shelf-life risks loss of functionality.
- Monitor the device and replace with a fresh dressing if it becomes saturated with fluid.
- For use up to 24 hours.


Reporting

Any serious incident that has occurred in relation to this device should be reported to the manufacturer (vigilance@safeguardmedical.com) and applicable competent authority.


Instructions for use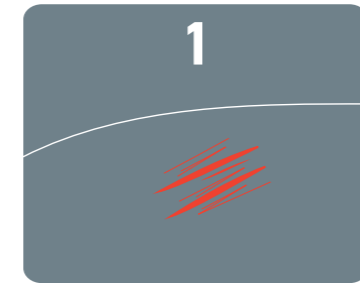

Before use, clean around acute wound. Note that excessively greasy skin will reduce the ability of the adhesive to function.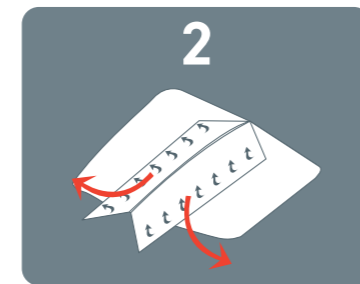

Remove the dressing from its packaging and unfold. Align the device so that it is directly over the wound and begin to peel back the plastic butterfly liners. If the wound cannot be covered by a single Corius Seal™ dressing, multiple dressings can be used by placing them edge to edge.

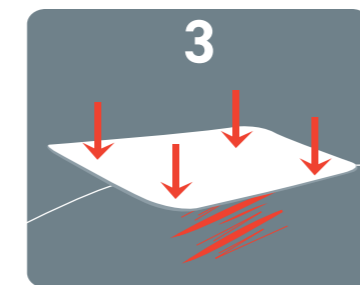Whilst removing the butterfly liners, place the exposed adhesive surface onto the wound. Gently smooth down the dressing's adhesive area to ensure that the dressing is properly attached.

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